
# INSTRUCTIONS FOR USE

## 3.0 ml Reservoir

THINset™ Model 103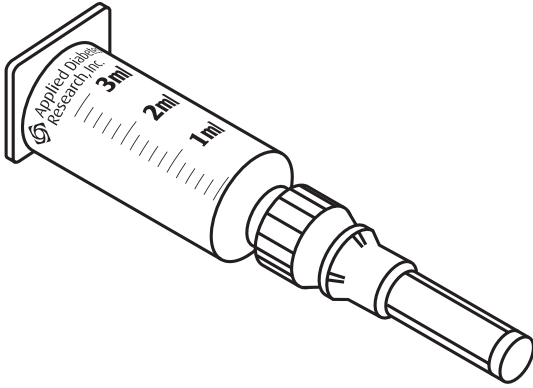

Applied  
Diabetes  
Research, Inc.

### NORTH AMERICA

Carrollton, Texas 75006

800.304.7293

972.241.1884

### EUROPE

CEpartner4U BV

Esdoornlaan 13

3951 DB Maarn



The Netherlands

+31 (0) 6-516.536.26

TSR-0000-0103 Rev A

	Expiration Date Fecha de vencimiento Date de péremption Data di scadenza Verfallsdatum 到期時間 到期時間
	Do Not Reuse No reutilizar À usage unique Monouso Nicht wiederverwenden 請勿重複使用 請勿重複使用
	Read Instructions for Use Lea las instrucciones de uso Lire le mode d'emploi Leggere le istruzioni per l'uso Vor dem Gebrauch die Gebrauchsanweisung lesen 請閱使用說明 請閱使用說明
	Lot Number Lote número Numéro de lot Numero di lotto Chargennummer 批号 批號
	Sterilized with Ethylene Oxide Esterilizado con óxido de etileno Stérilisé à l'oxyde d'éthylène Sterilizzato con ossido di etilene Mit Ethylenoxid sterilisiert 已用环氧乙烷(Ethylene Oxide)消毒 業已用環氧乙烷(Ethylene Oxide)消毒
	Reference/Model number Referencia/Número de modelo Numéro de référence/modèle Numero di riferimento/modello Referenz-/Modellnummer 參考號碼/型号 參考號碼/型號
	Open here Abrir aquí Ouvrir ici Aprire qui Hier öffnen 在此处拆开 於此處拆開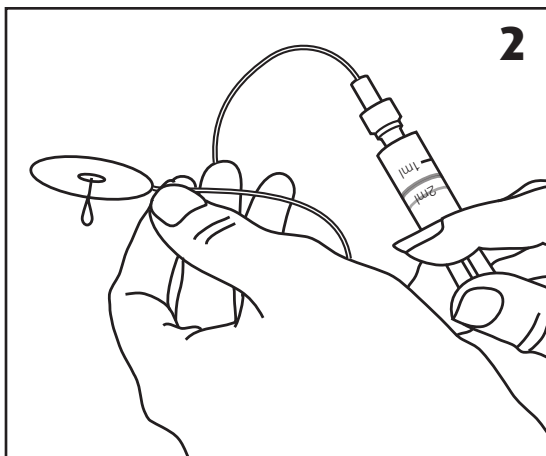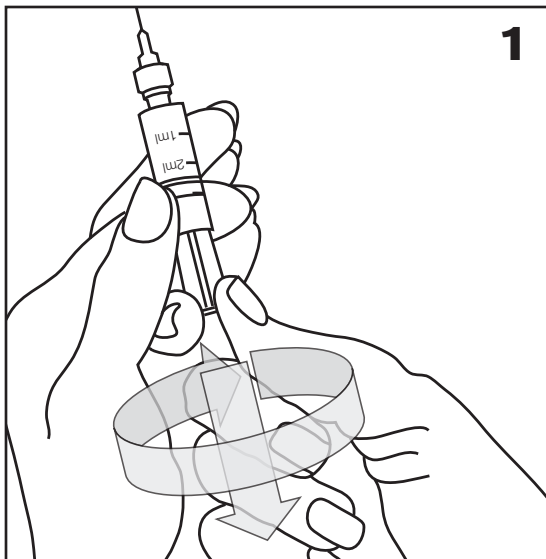

## 3.0 ml Reservoir

### THINSet™ Model 103

#### STERILE DEVICE

This device is sterile unless the package has been opened or damaged. This is a single-use item only.

#### USE WITH OTHER DEVICES

The 3.0 ml medication reservoir is intended for use with the MiniMed® 506, 507, 507C and 508 infusion pumps.

#### INDICATIONS FOR USE

The ADR THINSet™ 3.0 ml medication reservoir is intended for the subcutaneous infusion of medicine, including insulin, from an external infusion pump.

#### CONTRAINDICATIONS

The reservoir is not intended nor indicated for use with blood or blood products.

#### PRECAUTIONS

Always check for leaks after changing the reservoir and infusion set. Wrap a tissue around all Luer connections and check for moisture.

#### PRESCRIPTION DEVICE

Caution: U.S. law restricts this device to sale by or on the order of a physician.

If you require additional information regarding use of this reservoir, contact your health care professional or the Clinical Services Department at:

**North America**  
Carrollton, Texas 75006  
800.304.7293  
972.241.1884

**Europe**  
CEpartner4U BV  
Esdoornlaan 13  
3951 DB Maarn  
The Netherlands  
+31 (0) 6-516.536.26

## Instructions for Use

1. To ensure proper distribution of lubricant in the barrel of the reservoir, pull back on the plunger while rotating it at the same time (see Figure 1). It is recommended to do this process at least twice before filling the reservoir with insulin or other medication.
2. Using proper aseptic technique, fill a new reservoir with the appropriate amount of medication. Include approximately 0.25 ml (25 units for U-100 insulin) extra to prime the infusion set.
3. Attach a new infusion set to the reservoir.
4. Prime the infusion set by slowly advancing the reservoir plunger until the infusion set is filled and medication drips from the needle (see Figure 2).

**NOTE: Dispose of the needle safely.**

**NOTE: Be certain not to leave large (greater than 1/4 inch) bubbles in either the reservoir or infusion set as they can result in reduced medication delivery. Generally, any bubble larger than the head of a pin should be eliminated.**

5. Check for leaks at the junction of the reservoir and infusion set.

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